

## TECHNICAL DATA

### Contaflex GM3

#### Special Attributes

Nature: Terpolymer based on glycerol methacrylate

The GM3 range of materials is available in two water contents 49% and 58%, based on high purity glycerol methacrylate, these materials are remarkably easy to machine. Resultant lenses are extremely durable, resilient to protein build up and can be used for a variety of wearing regimens.

#### Material Characteristics

PROPERTY	CONTAFLEX GM3 49%	CONTAFLE GM3 58%
Classification	Filcon I 2	Filcon II 2
USAN	Acofilcon B	Acofilcon A
Ionic or Non-Ionic	Non-Ionic	Non-Ionic
Swell Factor at 20°C	1.29	1.38
Water Content at 20°C	50 % by refractometer 48 % by weight	59 % by refractometer 57.5 % by weight
Refractive Index	1.522 Dry 1.416 Hydrated	1.523 Dry 1.402 Hydrated
Light Transmission	>94 %	>95 %
Oxygen Permeability (ISO) at 35° [barrers]	15.9	25.5
Density	1.226 g/cm <sup>3</sup> Dry 1.142 g/cm <sup>3</sup> Hydrated	1.222 g/cm <sup>3</sup> Dry 1.103 g/cm <sup>3</sup> Hydrated
Tensile Strength	0.6 MPa	0.4 MPa
Elongation to Break	212 %	196 %
Shore D Hardness	87	88
Handling Tints	Clear, Blue	Clear, Blue
Diameter	12.70 mm	12.70 mm
Thickness	5.00 mm	5.00 mm

## TECHNICAL DATA

### Machining Recommendations

LATHE TYPE	RPM	BULK CUT	FINE CUT
Manual	7,000	0.50 - 0.70 mm	0.05 mm
CNC	7,000	0.50 - 0.70 mm	0.05 mm
Air Bearing	8,000	0.50 - 0.70 mm	0.03 - 0.05 mm

### Diamond Tooling

TOOLING	RADIUS
Rough	0.30 - 0.50 mm
Fine	0.30 - 0.50 mm

### Environment Control

For best manufacturing conditions Contamac recommends 19-23°C with a relative humidity of 45% - 60%

### Polishing

The recommended polishing compound is Contapol 2 with a spindle speed of 3,500 rpm and weight of 240gms. With the above machining recommendations polishing should require a maximum of 30 seconds.

### Hydration

Hydration of GM3 materials is best performed in buffered saline with a PH of 6.8 - 7.3.

### Manufacturing Consumables

- High melt blocking wax
- High melt blocking beewax
- Kemdent Blocking Wax
- Water soluble wax
- Contapol 1 polishing Compound
- Contapol 2 polishing Compound
- Cotton polishing cloth
- Delrin polishing cup
- Delrin polishing sponge
- Roller sponges

## CASE STUDY

### Accessories

- Tweezers
- Hydratation baskets
- Glass Vials
- Stoppers
- Aluminium top seals
- Flip tear up seals
- Shrink wrap sleeves
- Double soft lens mailers
- Net vial sleeves
- DMV soft lens remover
- Lens catch mat

The values quoted on this data sheet are indicative of the typical values that can be obtained for the material. For the complete specification, please contact Contamac at +44 (0)1799 514800 or [sales@contamac.co.uk](mailto:sales@contamac.co.uk). Unauthorised copy or reproduction of the above information is strictly prohibited without the prior consent of Contamac Ltd.

Version: 1 - 210408

Contamac Ltd  
Carlton House, Shire Hill  
Saffron Walden, Essex  
CB11 3AU

T. +44 (0)1799 514800  
E. [sales@contamac.co.uk](mailto:sales@contamac.co.uk)  
[www.contamac.com](http://www.contamac.com)