

CARE REGIME

DEFINITIVE

Multi Purpose Solutions

Multi-purpose solutions are the preferred choice for use with the Definitive range of materials. Stability studies across a range of the most readily available products have shown compatibility with all products tested.

Hydrogen Peroxide Solutions

Definitive 74

Peroxide solutions may be used with Definitive 74, but exposure to peroxide should be limited by using a disc-based system preferably. Should a tablet neutraliser be used, it is recommended that the tablet be added at the same time as the solution.

Systems that typically use a neutralising solution and allow lenses to remain in contact with the fully active, un-neutralised solution for extended periods should be avoided. If such systems are used then there is a risk that the stability of the lenses will be compromised.

Definitive 50 & 65

Both Definitive 50 and Definitive 65 are dimensionally stable with all peroxide systems, including those requiring overnight storage in un-neutralised hydrogen peroxide, and should be the materials of choice if prolonged exposed to hydrogen peroxide is likely to be required. For maximum compliance and in cases of lipid deposition we recommend cleaning with a surfactant cleaner (see also case study).

Enzymatic Cleaners

Enzymatic cleaners may be used, if required, with the Definitive range of materials.

Contamac does not recommend the use of alcohol based cleaners with the Definitive range of materials.

Please note that we make no comments on the efficacy of any solutions for lens cleaning, disinfection or general performance. These recommendations and comments only refer to compatibility with the lens material. For efficacy claims, please refer to solution manufacturers' literature.

The values quoted on this data sheet are indicative of the typical values that can be obtained for the material. For the complete specification, please contact Contamac at +44 (0)1799 514800 or sales@contamac.co.uk. Unauthorised copy or reproduction of the above information is strictly prohibited without the prior consent of Contamac Ltd.

Version: 1 - 210408

Contamac Ltd
Carlton House, Shire Hill
Saffron Walden, Essex
CB11 3AU

T. +44 (0)1799 514800
E. sales@contamac.co.uk
www.contamac.com